VULPES GOODS
BABYCARE

ELECTRIC BREAST PUMP

ELITE

**MANUAL • HANDLEIDING
ANLEITUNG • MANUAL**

WWW.VULPESGOODS.COM

ENGLISH

3

Product Information	4
Warnings	19
Cleaning & maintenance	22
FAQ	25
Product parameters	26

DEUTSCH

27

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


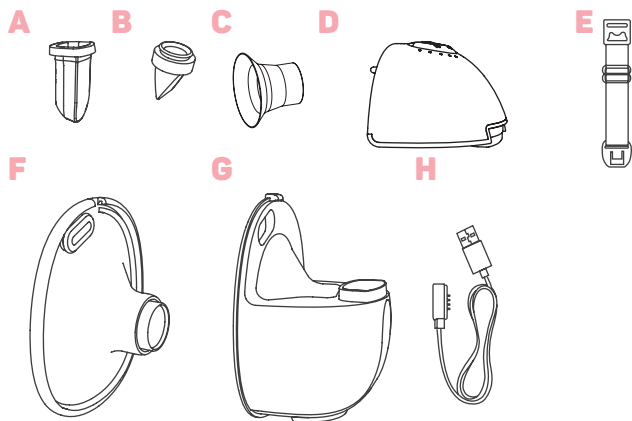
Assembling the Breast Pump

If you want to clean the pump before use, it's helpful to know how to properly assemble it afterward. Correctly assembling the pump and all its parts is crucial for optimal performance. Follow the steps carefully and make sure each part is placed correctly.



Step 1: Placing the Duckbill Valve

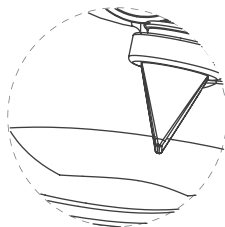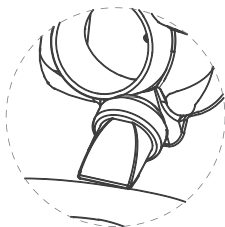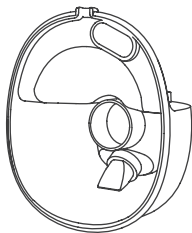Place the duckbill valve in the collection cup of the pump. Press the valve firmly to ensure that no milk can backflow. Make sure the valve is positioned horizontally for optimal performance.


Step 2: Placing the Gas Storage

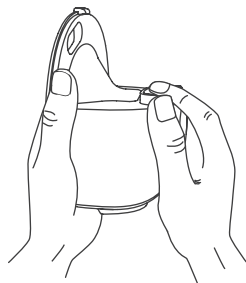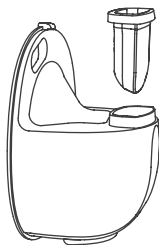Place the gas storage at the back of the collection cup with the bulging side facing the nipple. Press the gas storage firmly for a stable placement.


Step 3: Placing the Breast Shield

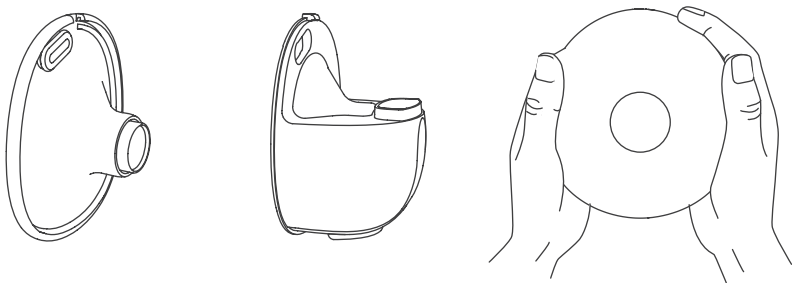
Place the breast shield onto the collection cup. Ensure that the breast shield fits completely onto the rim of the collection cup. The magnetic contact on the breast shield must be aligned with the magnetic contact on the motor. Press the edges with both hands to ensure a proper seal and prevent leakage.

Step 4: Placing the Motor

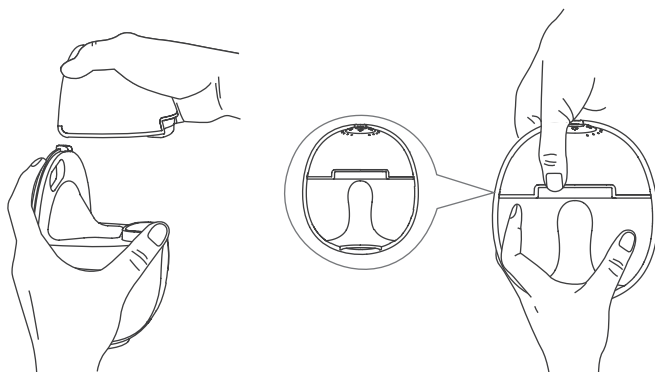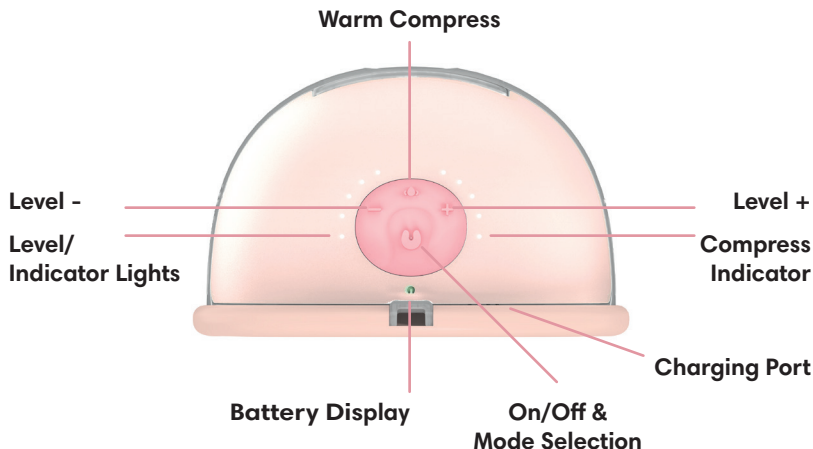Attach the motor to the back of the pump. Press the motor firmly onto the gas storage and make sure the magnetic contact aligns with the breast shield for proper functioning.

Warm Compress

Press once to turn on the warm compress. Press again to turn it off. The warm compress will only work if the pins on the breast shield are connected to the contact point on the motor. The warmth cannot be adjusted or set. It is possible to turn on the compress only and leave the pump mode off.

Note: The compress reaches a temperature of 40°C. This may also cause the motor to feel warmer.

Level + / Level -

Press once on the + button to increase the level.

Press once on the - button to decrease the level.

Level/Indicator Lights

The number of lights lit shows the set level. Use the + and - buttons to adjust the level. Level 1 is the lowest, and level 9 is the highest. Each level increases suction power.

Compress Indicator

When you press the warm compress button, the indicator will start blinking. Blinking means the warm compress is heating up. A steady light means the compress is being kept warm.

Charging Port

Use the included charger to charge the pump. Place and remove the charger carefully. The pump cannot be used while charging; the motor will not turn on.

On/Off & Mode Selection

Hold the button for 2 seconds to turn the pump on. Hold it again for 2 seconds to turn it off. Press the button once to select the mode when the pump is on.

The pump has 3 modes: massage, pump, and lactation mode. If the level lights blink quickly while pumping, the massage mode is active. If the level lights are on continuously while pumping, the expression mode is active. If the level lights blink briefly, the lactation mode is selected.

Battery Display

The battery display shows the status of the battery.

During Use

Constant green:	Battery is fully charged
Flashing red:	Battery is almost empty
Constant red:	Battery is empty

During Charging

Constant green:	Battery is charged and ready to use
Flashing green:	Battery is almost charged
Flashing red:	Battery is charging

Memory Mode

The pump remembers the last selected level in the mode. When switching between modes, the pump will continue at the last used level.

Pump Mode (Sucking mode) (9 levels) - ☺

Use the pump mode for full expression. Each level provides varying strengths during pumping. If the level lights are on continuously while pumping, the expression mode is active.

Massage Mode (9 levels) - ♥

It is recommended to massage the breasts for 2 - 5 minutes before pumping. This helps increase milk yield and comfort during pumping. If the level lights blink quickly while pumping, the massage mode is active.

Lactation Mode (9 levels) - Ⓐ

The mode combines massage and pumping for optimal comfort. Level 1-5 concerns the massage mode and then automatically switches to level 6-7 in the pump mode. If the level lights blink slowly while pumping, the lactation mode is active.

Levels 1-5: Massage to stimulate the breasts and encourage milk flow. The levels increase automatically every 15 seconds.

Levels 6-7: Pump mode to efficiently express milk. The automatic operation stops at level 7 to avoid discomfort or excessive pressure on the nipples.


Note: Levels 8 and 9 can only be selected manually. These higher levels provide extra suction power. The maximum level is 9, but the mode will never automatically go higher than level 7.

Insert Funnel

Choosing the correct insert funnel is important for optimal results. It ensures a better fit and prevents leakage. The table helps you make the right choice.

Nipple diameter before pumping (mm)	Nipple diameter after pumping (mm)	Suggested size(mm)
10-11-12	12-13-14	15mm
12-13-14	14-15-16	17mm
14-15-16	16-17-18	19mm
17-18-19	18-19-20	21mm
20-21-22	21-22-23	24mm

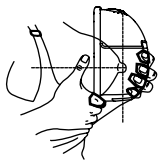Placement of the breastpump

Press the breast shield against the breast, ensuring there are no gaps. Make sure the nipples are properly centered and keep the breast shield upright.

Once the pump is in position, remain upright or lean slightly forward with your upper body. Make sure the pump is positioned properly and start pumping.

When you're done pumping, lean forward to detach the pump. Lean slightly forward to prevent milk from spilling.

Bra - strap

Adjust the length of the bra with the straps to customize the fit. This can make pumping more comfortable.

The end of the bra strap can be attached to the nursing bra.

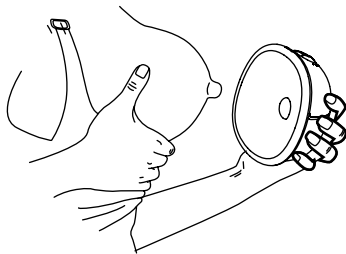
The length of the bra strap can be adjusted by pulling the bra strap tighter.

Pouring the milk

When you're done pumping, remove the breast pump from the breast. Place the pump upright on the base and remove the motor. Make sure the opening of the collection container is aligned with the opening of the bottle to prevent spilling breast milk.



Step 1: Disconnecting from the Breast

When you're done pumping, press the power button to turn off the pump. Break the seal between the breast and the pump with your finger, and gently remove the pump from your breast.

Step 2: Tilting and Pouring

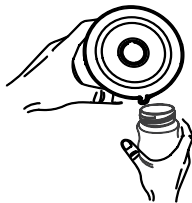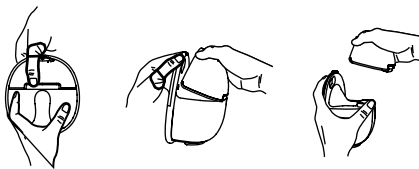Remove the pump motor from the collection container by holding the motor horizontally and first detaching the magnetic contact. Pour the milk out, ensuring the groove is properly aligned.

Connecting to the app

To make pumping even easier, you can connect the breast pump to the Tuya app via Bluetooth. With the app, you can turn the pump on, change the mode, set a timer, and activate the warm compress. Follow the steps below to use the app correctly.

Using the app

- With the app, you can turn the pump on and off, switch between modes, and activate or deactivate the warm compress.
- During operation, there may be a one-second delay before the action is executed.
- Always select the correct pump to operate. We recommend giving each pump a unique name. You can change the name using the pencil icon in the top right corner.
- If the pump is connected, it can still be operated manually.
- To disconnect the pump from the app: Ensure the motor is powered on and connected. Press and hold the power button until all 9 LED lights flash once. This will disconnect the Bluetooth connection. The pump can then be paired again.

Step 1: Connecting the Pump to the App

Ensure the pump is turned on and the massage mode is active. Open the app and click the orange button “Add Device.”

The app will automatically proceed to the next screen. Continue to step 2.

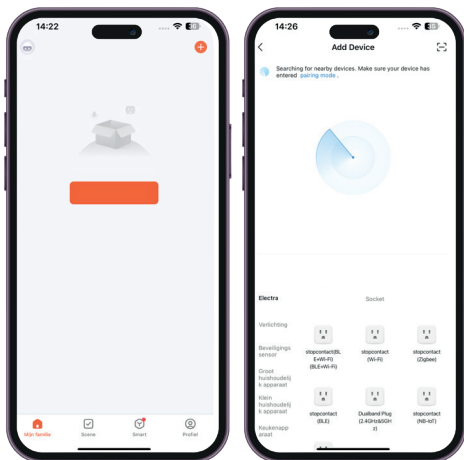
Alternative Pairing Method:

If pairing fails, there is a second method. Follow the steps below carefully:

Turn on the pump and select the massage mode.

Turn off the pump, wait 5 seconds, and press and hold the power button until the pump turns on and the level indicator starts flashing.


Keep holding the power button until the pump is detected by the app.

Step 2: Pairing the Pump with the App


Once the app detects the pump, it will appear in the device list. Click on the pump, and the app will automatically connect.

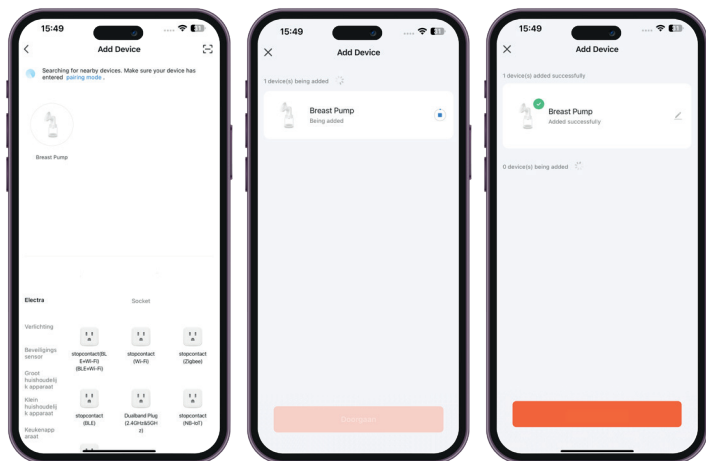When the green checkmark appears, the pump is successfully paired with the app. Click “Continue” to operate the pump and proceed to step 3.

Step 3: Operating the Pump from the App

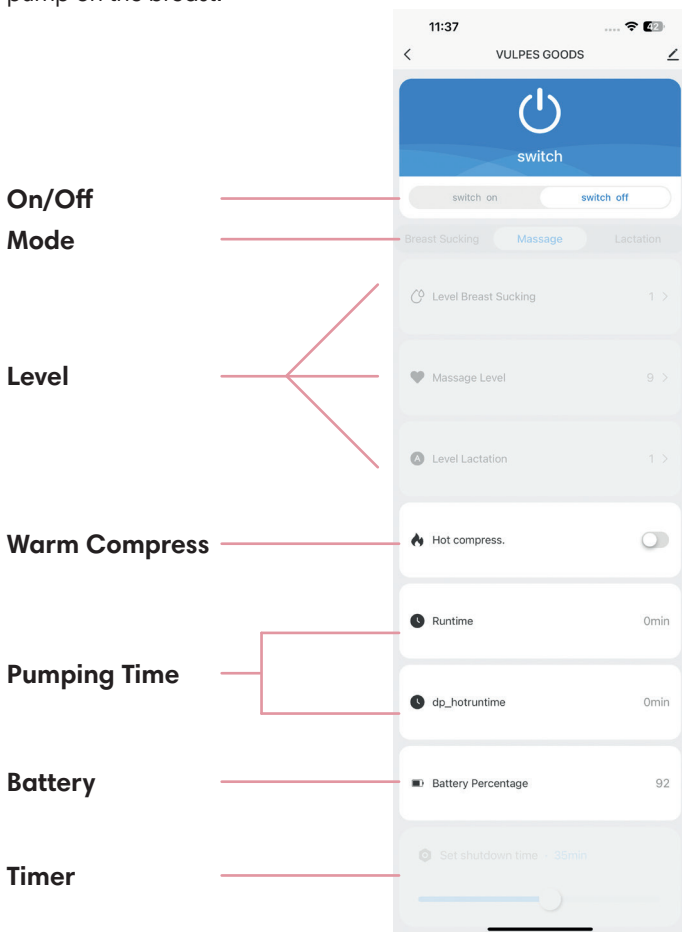Once the pump is paired, you can control the motor via the app. Read the instructions carefully and try out the functions first without placing the pump on the breast.

On/Off

Press “Switch on” to turn the motor on and press “Switch off” to turn it off. The large blue button is not operable.

Mode

Press the desired mode to activate it. The mode you did not select will appear in light gray and is not operable.

Level

After selecting a mode, the level will become available. By clicking on the level, a slider will appear, allowing you to adjust the setting. The unselected mode will appear in light gray and is not operable.

Warm Compress

Press once to turn the compress on. Press again to turn it off.

Pumping Time

Track the pumping time with the timer.

Battery

The battery display shows the status of the battery. The battery drains faster with frequent and rapid adjustments via the app.

Timer

Set the timer to the desired time. Once the time is reached, the pump will automatically turn off.

WARNINGS

1. Use

- Use the breast pump exclusively for expressing breast milk. Other uses are considered improper use of the product.
- Consult a doctor or lactation consultant before using the breast pump, especially if you have medical questions or experience issues with breastfeeding.
- Ensure that the suction strength and speed are correctly adjusted to prevent discomfort and potential damage to the nipples or breasts.
- Do not use the breast pump on irritated or damaged skin. Wait until the skin has healed before using the breast pump again.
- Do not use the breast pump while sleeping or resting. Stay alert and aware during pumping.
- Pumping at regular intervals helps to maintain a consistent milk supply.
- Vary the modes and settings to prevent habituation. This can improve output.

2. Safety and Maintenance

- Inspect the breast pump and all parts for damage or wear before each use. Do not use damaged parts. Contact us immediately for defective parts.
- Only use the included charging cable. Using an incorrect power source can cause damage.
- Keep the motor and adapter dry and away from water or other liquids.
- Store the breast pump and accessories out of reach of children and pets.

- Stop pumping and consult a healthcare provider if you experience pain or discomfort while using the breast pump.
- Ensure the motor remains free from dust and milk residue. This can affect the durability of the motor.
- Ensure the motor is always free from milk, water, dust, dirt, and residue, especially on the underside of the motor, as this can negatively impact the motor's performance and durability. Damage from milk, water, dust, dirt, residue, improper handling, or negligent cleaning and usage is not covered under warranty.
- Do not drop the motor. Impact damage can cause serious internal damage and is not covered under warranty.
- Regularly check the silicone parts (like valves) and replace them if necessary, as they can wear out and affect suction power.
- Do not use chemicals or aggressive cleaning agents on the parts. This can be harmful to the product and the user.
- Always place the motor with the buttons facing up. The motor should be standing and not lying down. Placing the motor down can affect the durability of the pump.

3. Cleaning and Storage

- Thoroughly wash your hands before assembling or using the breast pump.
- Clean the breast pump and accessories after each use according to the instructions in the manual to prevent infections.
- Do not put the parts in the dishwasher; this can cause damage that is not covered under warranty.
- Do not put the parts in the microwave sterilizer, as this may cause damage that is not covered under warranty.
- Store breast milk in clean bottles, bags, or well-sealed containers and at the correct temperature. Label these with the date of expression.
- Never heat breast milk in a microwave. Use a bottle warmer or warm water bath.
- Ensure all parts are completely dry before reassembling the breast pump to prevent electrical issues or mold growth.
- Do not use brushes or sharp objects to clean the valves, as this may cause damage. Do not pierce the valves and avoid applying excessive pressure.

4. Warranty and Contact

- In case of a defect, contact Vulpes Goods directly. Do not attempt to repair the device yourself and stop using it immediately.
- Water and impact damage are not covered under warranty. Keep the motor dry and do not drop it.
- Failure to follow the instructions, cleaning guidelines, and warnings may result in the warranty being voided. Engels

CLEANING & MAINTENANCE

Before cleaning and disinfecting, please remove all accessories and the motor. The pump motor contains electronics that should be kept dry. Use a clean cloth to wipe the pump motor. Any other cleaning method will be classified as misuse under the warranty.

**Clean the parts according to the instructions in the manual.
Do not use a dishwasher or microwave sterilizer.**

Breast pumps are personal care items intended for a single user only and should never be rented or shared. Immediately separate and wash all parts that come into contact with the breast and breast milk after use to avoid residue buildup and prevent bacteria growth.

Cleaning

Non-washable Parts:

The motor must never come into contact with milk, water, dust, dirt, and residue, especially on the underside of the motor. The pump must remain dry and clean. Use a clean, dry cloth to wipe the main pump unit. Keep the motor dry and free of residue. Any cleaning method outside of these instructions may void the warranty due to improper use.

Washable parts:

This includes the gasstorage, the duckbill valve, the insert funnel, the breastshield, and the collection cup. Cleaning the washable parts that come into contact with milk prevents the buildup of dried milk residue, which can promote the growth of harmful bacteria. This is essential for the safety of the product, you, and your baby.

Supplies needed:

Mild dish soap, clean discloth, clean wash basin & clean water

1. Rinse all separated parts that come into contact with the breast and breast milk in water at 25-30°C (77-86°F) to remove any milk residue.
2. Once rinsed, place the parts that have touched the breast or breast milk in a clean wash basin designated only for washing infant feeding equipment.
3. Let all washable parts soak in soapy water at 40-60°C (104-140°F) for 3-5 minutes. Wash each part with a clean dishcloth or sponge.
4. Place the cleaned parts on a clean surface or towel and allow them to air dry. Store dry parts in a clean, cool place when not in use. Do not store wet or damp parts.

Sanitizing

Non-washable parts:

The motor must never come into contact with milk, water, dust, dirt, and residue, especially on the underside of the motor. The pump must remain dry and clean. Use a clean, dry cloth to wipe the main pump unit. Keep the motor dry and free of residue. Any cleaning method outside of these instructions may void the warranty due to improper use.

Washable parts:

This includes the gasstorage, the duckbill valve, the insert funnel, the breastshield*, and the collection cup. Cleaning the washable parts that come into contact with milk prevents the buildup of dried milk residue, which can promote the growth of harmful bacteria. This is essential for the safety of the product, you, and your baby.

Supplies needed:

Clean pot for +100°C / +212°F water, clean water & tongs

1. Fill a pot with enough water to cover all parts. Bring the water to a boil. Place parts in water at +100°C / +212°F for 5-10 minutes, ensuring that the parts do not touch the sides of the pot or each other to avoid melting together.
2. Allow the water to cool and gently remove the parts from the water using tongs. Be cautious as both the water and the parts can be hot.
3. Place the sanitized parts on a clean surface or towel and allow them to air dry. Store dry parts in a clean, cool place when not in use. Do not store wet or damp parts.

* **Note:** Do not let the breast shield come into contact with the pan. If the breast shield touches the pan, it may melt.

**Replace washable parts after signs of wear and tear.
Replace washable parts every 2 to 4 months for hygiene.**

FAQ

Q. My breast pump has low suction?

A. The gas storage and pump unit are not properly installed in place. When the pump is installed in the gas storage unit, it must be fully depressed.

A. The duckbill valve is not installed in place or is damaged. Check the part for proper placement and possible damage.

Q. The milk flows back.

A. The gas storage is damaged or deformed, the milk flows back into the pump. In this case, stop using it immediately. Replace the gas storage to restore the connection.

A. The duckbill valve contains a crack. This allows the milk to flow back. Check the valve for cracks, damage and proper placement.

Q. Pumping hurts / pumping fails.

A. You may have a breast congestion or blocked milk duct. Ask a doctor to massage the breast.

A. Get the right breastplate size. Choosing the right size is important for a good fit.

A. The pump level used is too strong. Start the pumping session from the lower level and build up slowly.

Q. The milk remains on the breastplate.

A. Check that you are using the right breast shield reducer. Sometimes a size that is too small or too large can cause the milk not to flow properly.

Q. My breast pump is not charging properly?

A. Make sure you use the USB cable provided. Connect it to a suitable adapter with the correct voltage.

Q. The breast shield doesn't connect to the motor.

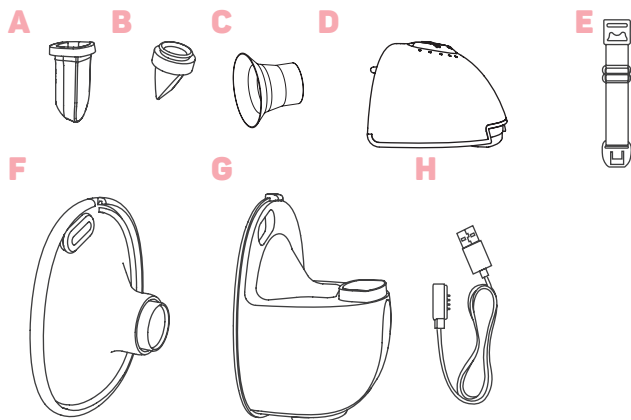
A. Place the pump in a well-fitting nursing bra; this will automatically press the breast shield against the motor to ensure proper contact.

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