VULPES GOODS

BABYCARE

SILVER NURSING CUPS

**MANUAL • ANLEITUNG
HANDLEIDING • MANUEL
MANUAL • MANUALE • MANUAL**

WWW.VULPESGOODS.COM

ENGLISH **4**

User Guide 5

Characteristics and Application 6

FAQ 7

DEUTSCH **8**

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FAQ 11

NEDERLANDS **12**

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Kenmerken en Toepassingen 14

FAQ 15

FRANCAIS **16**

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ENGLISH

USER GUIDE

- 1.** Squeeze a little breast milk into the nursing cups and place them over the nipples with the hollow side facing in.
- 2.** If desired, place a nursing pad over the nursing cups (Nursing pads help keep the cups in place and prevent leaking).
- 3.** Wear a well-fitting, stretchy bra that keeps everything securely in place.
- 4.** Rinse the nursing cups with warm water after daily use. If residue appears: make a paste of baking soda and water, gently rub it on the shields, and rinse thoroughly.

CHARACTERISTICS AND APPLICATION

- Keep the nursing cups out of reach of children.
- Do not use the nursing cups in combination with other nipple treatments. This may reduce their effectiveness.
- Do not use the nursing cups if you are allergic to silver.
- Discontinue use if skin irritation, itching, or redness persists.
- Do not use cleaning agents or abrasive materials for cleaning.
- The nursing cups are not suitable for dishwasher or microwave use.

FAQ

Q. Can nursing cups be used to prevent sore nipples?

A. Yes! We recommend using nursing cups as a preventive measure for sore, cracked, or bleeding nipples. Silver is a natural antimicrobial, anti-inflammatory, and healing material that is very effective in preventing nipple pain.

Q. Does the effectiveness of the nursing cups decrease over time?

A. No, the effectiveness of the nursing cups remains the same, no matter how long you use them!

Q. Is it normal for the nipple to be pushed inward?

A. No. If the nursing cups are placed correctly, they fit perfectly on the breast. If your nipple is being pushed inward, check if your bra is too tight and pressing on the areola.

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