VULPES GOODS
BABYCARE

ELECTRIC BREAST PUMP

PRO

**MANUAL • HANDLEIDING
ANLEITUNG • MANUAL**

WWW.VULPESGOODS.COM

ENGLISH

3

Product Information	4
Warnings	13
Cleaning & maintenance	16
FAQ	19
Product parameters	20

DEUTSCH

21

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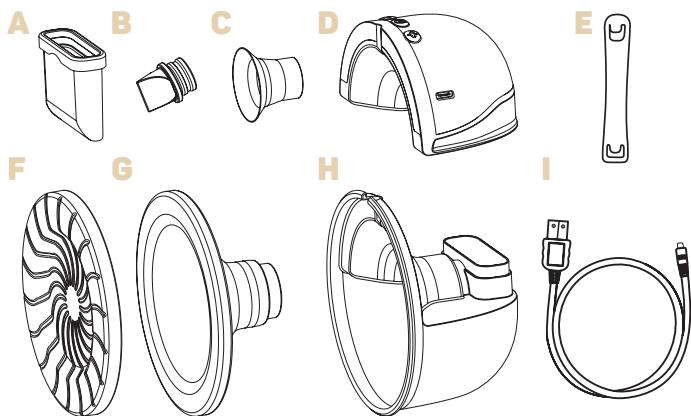
ENGLISH

PRODUCT INFORMATION


	NAME OF PARTS	MATERIAL	HEAT-RESISTANCE TEMPERATURE
A	Gasstorage	Silicone	-20~180 °C
B	Duckbill valve	Silicone	-20~180 °C
C	Insert funnel	Silicone	-20~180 °C
D	Engine / Pump unit	ABS/Silicone	/
E	Adjustment buckle	/	/
F	Sealing cap	ABS	/
G	Breastshield	Silicone	-20~180 °C
H	Collection cup	Silicone	-20~180 °C
I	Charging cable	/	/

Assembly of the breast pump

Should you wish to clean the pump extra before use, it is useful to know how to assemble it correctly next. Assembling the breast pump and all its parts correctly is crucial for optimal use. Follow the steps carefully and make sure each part is correctly positioned.

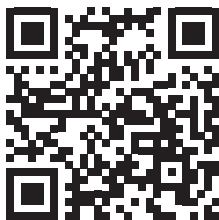
When you scan the QR code, you will be directed to the instructional video. This will show you the best way to assemble, pour, and clean the pump. On the next page, you will find step-by-step instructions on how to assemble the parts and what to pay attention to.

Note: Before using the breast pump, watch the video for proper installation, cleaning, and maintenance. Always carefully watch the video and read the manual. This will help you use the breast pump effectively.


Step 1: Installation of the gas storage

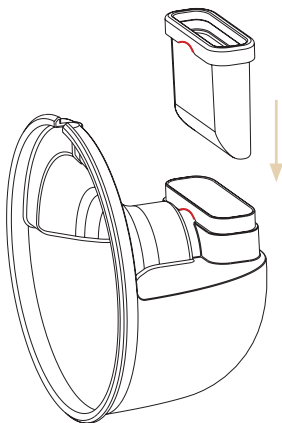The gas storage is placed at the back of the collection tray. Make sure the bottom is correctly fitted with the **rounding** towards the collection tray.


Step 2: Positioning the valve

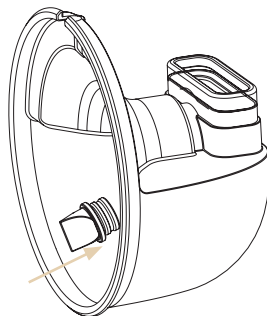The valve is placed in the receptacle of the flask. Make sure the valve is fully pressed on. This will prevent the milk from flowing back.


Step 3: Placement of the breast shield

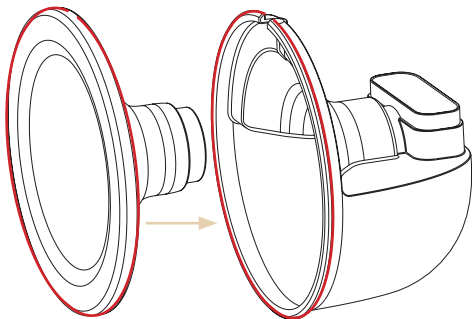The breast shield is placed on the collection tray. It is crucial that the breast shield is fully pressed on and fits over the **edge** of the receptacle.


Step 4: Installing the motor

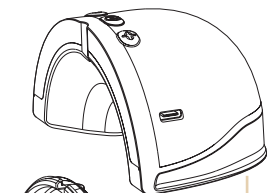
The motor is attached to the back of the flask. Press the motor firmly onto the gas storage.

Step 5: Inserting the breast shield reducers

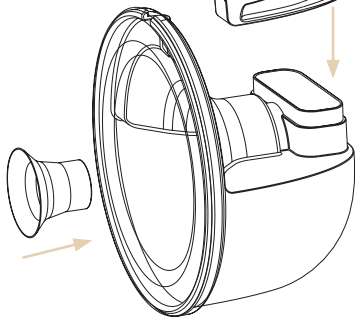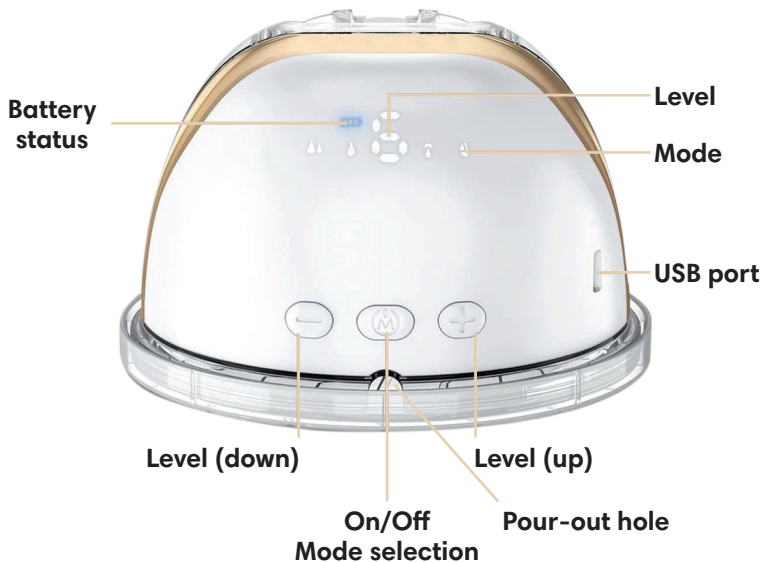Choose the right size breast shield reducer and insert it into the breast shield. Press the breast shield reducer firmly to prevent leakage.

Battery status:

Indicates the % battery remaining and charging status.

Low battery:

0-10% Battery turns red

Medium battery:

11-75% Battery turns white

Full battery:

76-100% Battery turns blue

When the battery needs charging the icon flashes red.

When the battery is charging, the icon flashes blue/white.

When the battery is fully charged the icon turns blue.

(Battery life depends on the modes and levels used)

Level + or -:

Use this to select the level used to pump. Depending on the selected mode, 1 - 6 levels are available. Each level up increases the suction power. (Always start at level 1 and build up to the desired level)

On/off & mode selection

Press and hold the button for 2 seconds to switch the flask on. Press again to select the mode. Press and hold the button for 2 seconds to turn the flask off.

USB port

Use the supplied charger to charge the flask. Insert and remove the charger carefully. Do not use excessive force, this may cause damage.

Mode

The display shows the current mode, each mode has a different function. Using the **M** button, you can change the mode.

Level

The display shows the current level; each level increases the suction power. Using the + and - button, the level can be changed.

Timer

The pump has a built-in timer. After 30 minutes of pumping, the pump automatically goes into 'sleep mode'. You can turn the pump back on after this.

Memory mode

The flask remembers the last selected level in the mode. If you switch between modes, the flask will continue at the last stopped level.

Massage/stimulation mode (1 - 5 levels)

It is advised to massage the breasts for 2 minutes before pumping. This improves the yield and convenience during pumping.

Expression/pump mode (1 - 6 levels)

Use expression/pump mode to pump fully. Each level offers different strengths while pumping.

2-in-1 mode (1 - 6 levels)

Can be very helpful for sensitive breasts and supports milk yield in a short time.

Double Frequenton (1 - 4 levels)

Can be very helpful for shortened or retracted nipples.

Note: Build up the levels slowly in each mode. Start at level 1 and build up to the desired level.

Breast shell reducers

Choosing the right breast shield reducers is important for optimal results. It ensures a better fit and prevents leaks. The table below will help you make the right choice.

Nipple diameter for pumping (mm)	Nipple diameter after pumping (mm)	Suggested size (mm)
14-15-16	16-17-18	19mm
17-18-19	18-19-20	21mm
20-21-22	21-22-23	24mm
24-25-26	25-26-27	28mm

Inserting the pump

Bend forward slightly and centre the nipple in the flange. Make sure the pump is tight against the breast.

Once the pump is in position, stay upright with the upper body. Make sure the pump is sitting up and start pumping.

Once you have finished pumping, lean forward to release the pump. Bend forward slightly to avoid spilling milk.

Bra - strap

Adjust the length of the bra with the straps. This can make pumping more comfortable.

The end of the bra strap can be linked to the nursing bra.

The bra strap length can be adjusted by pulling the bra strap tighter.

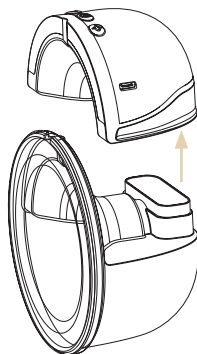
Pouring out

Pouring out the milk is easy via the pouring hole at the top of the collection tray. Always pour carefully without the motor sitting on the collection tray. This reduces the risk of moisture and milk damage.

You can refer to the instruction video for the correct way of pouring. The images below show step-by-step how to prepare the flask for optimal pouring.


Step 1: Remove the motor.

Do this carefully to avoid leaks. Leave the gas storage in place. Otherwise, there is a risk of a leakage.

Step 2: Tilting and pouring

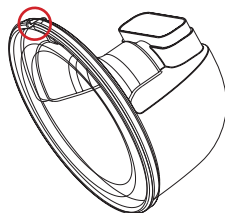Hold the breast pump by the breast shield, this is easiest to pour and gives control. As pouring improves, rotate the breast pump further and further until the receptacle is empty.

WARNINGS

1. Use

Use the breast pump exclusively for expressing breast milk. Other uses are considered improper use of the product.

Consult a doctor or lactation consultant before using the breast pump, especially if you have medical questions or experience issues with breastfeeding.

Ensure that the suction strength and speed are correctly adjusted to prevent discomfort and potential damage to the nipples or breasts.

Do not use the breast pump on irritated or damaged skin. Wait until the skin has healed before using the breast pump again.

Do not use the breast pump while sleeping or resting. Stay alert and aware during pumping.

Pumping at regular intervals helps to maintain a consistent milk supply.

Vary the modes and settings to prevent habituation. This can improve output.

2. Safety and Maintenance

Inspect the breast pump and all parts for damage or wear before each use. Do not use damaged parts. Contact us immediately for defective parts.

Only use the included charging cable. Using an incorrect power source can cause damage.

Keep the motor and adapter dry and away from water or other liquids.

Store the breast pump and accessories out of reach of children and pets.

Stop pumping and consult a healthcare provider if you experience pain or discomfort while using the breast pump.

Ensure the motor remains free from dust and milk residue. This can affect the durability of the motor.

Ensure the motor is always free from milk, water, dust, dirt, and residue, especially on the underside of the motor, as this can negatively impact the motor's performance and durability. Damage from milk, water, dust, dirt, residue, improper handling, or negligent cleaning and usage is not covered under warranty.

Do not drop the motor. Impact damage can cause serious internal damage and is not covered under warranty.

Regularly check the silicone parts (like valves) and replace them if necessary, as they can wear out and affect suction power.

Do not use chemicals or aggressive cleaning agents on the parts. This can be harmful to the product and the user.

Always place the motor with the buttons facing up. The motor should be standing and not lying down. Placing the motor down can affect the durability of the pump.

3. Cleaning and Storage

Thoroughly wash your hands before assembling or using the breast pump.

Clean the breast pump and accessories after each use according to the instructions in the manual to prevent infections.

Do not put the parts in the dishwasher; this can cause damage that is not covered under warranty.

Store breast milk in clean bottles, bags, or well-sealed containers and at the correct temperature. Label these with the date of expression. Never heat breast milk in a microwave. Use a bottle warmer or warm water bath.

Ensure all parts are completely dry before reassembling the breast pump to prevent electrical issues or mold growth.

Do not use brushes or sharp objects to clean the valves, as this may cause damage. Do not pierce the valves and avoid applying excessive pressure.

4. Warranty and Contact

In case of a defect, contact Vulpes Goods directly. Do not attempt to repair the device yourself and stop using it immediately.

Water and impact damage are not covered under warranty. Keep the motor dry and do not drop it.

Failure to follow the instructions, cleaning guidelines, and warnings may result in the warranty being voided.

CLEANING & MAINTENANCE

Before cleaning and disinfecting, please remove all accessories and the motor. The pump motor contains electronics that should be kept dry. Use a clean cloth to wipe the pump motor. Any other cleaning method will be classified as misuse under the warranty.

Breast pumps are personal care items intended for a single user only and should never be rented or shared. Immediately separate and wash all parts that come into contact with the breast and breast milk after use to avoid residue buildup and prevent bacteria growth.

Cleaning

Non-washable Parts:

The motor must never come into contact with milk, water, dust, dirt, and residue, especially on the underside of the motor. The pump must remain dry and clean. Use a clean, dry cloth to wipe the main pump unit. Keep the motor dry and free of residue. Any cleaning method outside of these instructions may void the warranty due to improper use.

Washable parts:

This includes the gasstorage, the duckbill valve, the insert funnel, the sealing cap, the breastshield, and the collection cup. Cleaning the washable parts that come into contact with milk prevents the buildup of dried milk residue, which can promote the growth of harmful bacteria. This is essential for the safety of the product, you, and your baby.

Supplies needed:

Mild dish soap, clean discloth, clean wash basin & clean water

1. Rinse all separated parts that come into contact with the breast and breast milk in water at 25-30°C (77-86°F) to remove any milk residue.
2. Once rinsed, place the parts that have touched the breast or breast milk in a clean wash basin designated only for washing infant feeding equipment.
3. Let all washable parts soak in soapy water at 40-60°C (104-140°F) for 3-5 minutes. Wash each part with a clean dishcloth or sponge.
4. Place the cleaned parts on a clean surface or towel and allow them to air dry. Store dry parts in a clean, cool place when not in use. Do not store wet or damp parts.

Sanitizing

Non-washable parts:

The motor must never come into contact with milk, water, dust, dirt, and residue, especially on the underside of the motor. The pump must remain dry and clean. Use a clean, dry cloth to wipe the main pump unit. Keep the motor dry and free of residue. Any cleaning method outside of these instructions may void the warranty due to improper use.

Washable parts:

This includes the gasstorage, the duckbill valve, the insert funnel, the sealing cap, the breastshield, and the collection cup. Cleaning the washable parts that come into contact with milk prevents the buildup of dried milk residue, which can promote the growth of harmful bacteria. This is essential for the safety of the product, you, and your baby.

Supplies needed:

Clean pot for +100°C / +212°F water, clean water & tongs

1. Fill a pot with enough water to cover all parts. Bring the water to a boil. Place parts in water at +100°C / +212°F for 5-10 minutes, ensuring that the parts do not touch the sides of the pot or each other to avoid melting together.
2. Allow the water to cool and gently remove the parts from the water using tongs. Be cautious as both the water and the parts can be hot.
3. Place the sanitized parts on a clean surface or towel and allow them to air dry. Store dry parts in a clean, cool place when not in use. Do not store wet or damp parts.

**Replace washable parts after signs of wear and tear.
Replace washable parts every 2 to 4 months for hygiene.**

FAQ

Q. My breast pump has low suction?

A. The gas storage and pump unit are not properly installed in place. When the pump is installed in the gas storage unit, it should be fully depressed.

A. The duckbill valve is not installed in place or is damaged. Check the part for correct placement and possible damage.

Q. Milk flows back.

A. If the gas storage is damaged or deformed, milk will flow back into the pump. In this case, stop using it immediately. Replace the gas storage to restore the connection.

A. The duckbill valve contains a crack. This gives the milk a chance to flow back. Check the valve for cracks, damage and proper placement.

Q. Pumping hurts/pumping does not work.

A. You may have a breast congestion or blocked milk duct. Ask a doctor to massage the breast.

A. Get the right breastplate size. Choosing the right size is important for a proper fit.

A. The pump level used is too strong. Start the pumping session from the lower level and build up slowly.

Q. Milk stays on the breastplate.

A. Check that you are using the right breast shield reducer. Sometimes a size that is too small or too large can cause the milk not to flow properly.

Q. My breast pump is not charging properly?


A. Make sure you use the USB cable provided. Connect it to a suitable adapter with the correct voltage.

PRODUCT PARAMETERS

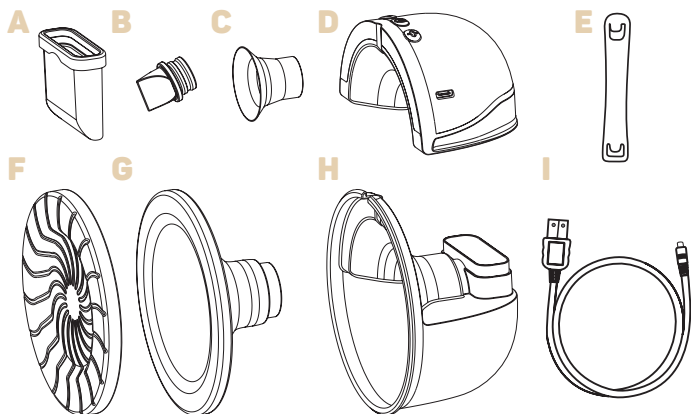Model:	VG-586
Weight (1 breast pump):	260Gr
Product size:	11.5cm x 7cm x 11.5cm (LxWxH)
Rated output:	5W
Input:	100-240V, 50/60Hz
Output power:	5V  1A
Suction strength:	≤ -0.035 MPA
Battery capacity:	1500 mAh
Modes:	4
Collection tray capacity:	150ML

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