VULPES GOODS
BABYCARE

ELECTRIC NASAL ASPIRATOR

**MANUAL • ANLEITUNG
HANDLEIDING • MANUEL
MANUAL • MANUALE • MANUAL**

WWW.VULPESGOODS.COM

ENGLISH	4
The Device	5
User Guide	8
Characteristics and Application	13
Product parameters	17
DEUTSCH	18
Das Gerät	19
Benutzerhandbuch	22
Eigenschaften und Anwendung	27
Produkt parameter	31
NEDERLANDS	32
Het apparaat	33
Gebruiksaanwijzing	36
Kenmerken en Toepassingen	41
Product Parameters	45
FRANCAIS	46
Le dispositif	47
Manuel de l'utilisateur	50
Caractéristiques et applications	55
Paramètres du produit	59

ESPAÑOL **60**

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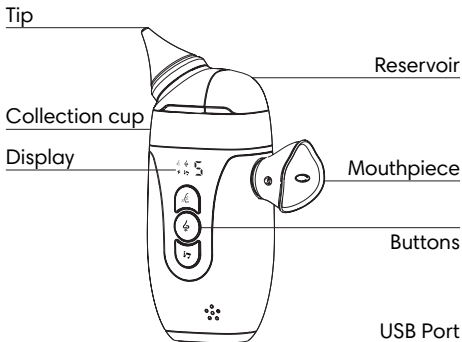
THE DEVICE

Reservoir

The reservoir is intended for the spray function. It is filled with water or a saline solution to moisten nasal mucus.

Mouthpiece

Place the mouthpiece on the mouth to ensure proper positioning during nebulization.

Buttons

The buttons are used to control all functions of the device. Press and hold a button for 2 seconds to turn the function on or off. Each press increases the selected level, if applicable.

Spray Mode

The spray mode has one level and is used for misting. Make sure the reservoir is filled with water. This function will not work without water.

Suction Mode

The suction mode has 5 levels and is used for cleaning the nose. Empty the collection container in time to prevent blockage.

Music Mode

The music mode includes 3 different songs to make the cleaning process more comfortable.

USB port

The USB port (Type-C) is used to charge the device with the supplied charging cable. The icon flashes during charging. When fully charged, the icon lights up solid green.

Tip

The tip is used for the suction function. It is inserted into the nostril.

Collection cup

The collection cup holds the liquid used for nebulization and collects nasal mucus during the suction function. It is removable and can be cleaned after use.

Display

The display shows the selected mode and the selected suction level (level 1 to 5).

spray mode

suction mode

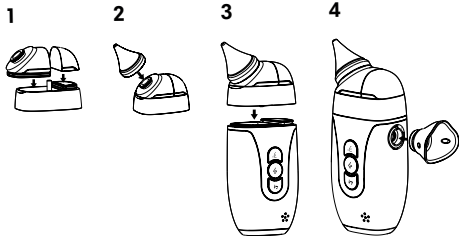
music mode

USER GUIDE

Ensure that the device is correctly assembled before use.

1. Place the lid onto the collection cup.
2. Attach the silicone nasal tip to the collection cup.
3. Insert the complete collection cup into the main unit of the device.
4. Then attach the silicone protective ring to the main unit.
5. Check that all parts are securely fastened. The device is now ready for use.

Using the Spray Function

1. Remove the spray cap from the device.
2. Fill the reservoir with suitable water, such as tap water, mineral water, saline solution, or softened water. Do not use hard water or fully purified (distilled) water.
3. Place the spray cap back onto the device.
4. Press and hold the spray mode button to start the spray.

Note:

The spray function stops automatically after 10 seconds. To continue spraying, the function must be reactivated. The spray function also stops automatically when the battery is almost empty.

Using the Suction Function

1. Attach the silicone suction tip to the device.
2. Ensure that the child is awake. Gently place the silicone tip into the nostril.
3. Do not insert the tip too deeply and do not press it against the inner wall of the nose. When used by adults, keep the head upright during insertion.
4. Press and hold the suction mode button for 2 seconds to start the device. Then select the desired level by pressing the button.

Note:

Never use the suction function upside down, as the device will not operate.

Cleaning the Device

Important

The main unit of the nasal aspirator cannot be cleaned with water. For hygiene reasons, it is important to clean the device immediately after each use. Remaining nasal mucus in the tips, collection cup, or base may cause bacterial growth and becomes more difficult to remove once dried.

Cleaning the Removable Parts

1. Carefully remove the collection unit from the device.
2. Disassemble the collection unit and its components.
3. If residue is difficult to remove, the removable parts may be soaked in warm water with a small amount of mild soap before cleaning.

Cleaning the Main Unit

1. Carefully clean the connection area using a cotton swab.
2. Wipe the exterior of the device with a paper towel or dry cloth.
3. Use the supplied brush to remove residue from hard-to-reach areas or dried mucus.
4. If the device is dirty, a slightly damp cloth or cleaning wipe may be used to clean the exterior. Ensure that no water enters the device.

CHARACTERISTICS AND APPLICATION

- Use this product only for its intended purpose and as described in this manual. Do not use accessories or parts that are not recommended by the manufacturer.
- Do not expose the device to extreme temperatures, high humidity, or direct sunlight. Do not place or store the device where it may fall or come into contact with water, such as near a bathtub, shower, sink, or washbasin.
- Do not allow objects to enter openings, tubes, or other parts of the device. Ensure that ventilation openings are not blocked and do not place the device on soft surfaces such as a bed or sofa. Keep all openings free of dust, hair, and other debris.
- Do not use the device when drowsy or while sleeping. Discontinue use immediately if irritation, excessive heat, irregular operation, or visible damage to the device occurs.

- Do not use the device in cases of nosebleeds, acute injuries to the head, face, or neck, blood clotting disorders, or other bleeding conditions. Consult a medical professional in case of doubt.
- Insert the tip only as far as necessary to create a gentle and comfortable seal around the nostril. Never force the tip.
- For the spray system, the use of soft water, mineral water, or bottled water is recommended. Hard water is not recommended, as limescale buildup may cause damage to the device.
- Clean the spray or nebulization components after each use, especially if hard water has been used, to prevent clogging and damage.
- Do not use pure distilled water, as it may reduce nebulization performance and cause clumping.
- It is normal for small water droplets to appear during filling due to pressure differences. Gently shake the device to allow it to function normally.

- Before each use, ensure that the device is complete, clean, and undamaged.
- Use the device in a clean, quiet, and well-ventilated environment to prevent interference and contamination.
- Empty the water reservoir when the device is not in use.
- Clean all parts after each use and disinfect them regularly to prevent bacterial growth.
- The device may only be operated by an adult. Children must not use the device independently.
- When the device is not in use, it must be kept out of the reach of children.
- The device is intended only for simple nasal cleaning. If discomfort or symptoms persist, contact with a medical professional is recommended.
- During use, the child must continue to breathe normally. The child must not hold their breath, close the mouth, or swallow during use, as this may cause pressure imbalance in the ears due to the suction force of the device.


- The device must be cleaned carefully after each use.
- Do not clean the device with detergents, chemicals, or aggressive liquids.
- Never immerse the device in water. The main unit of the nasal aspirator must not be cleaned with water.
- Do not place the device in a dishwasher, sterilizer, or similar appliance.

PRODUCT PARAMETERS

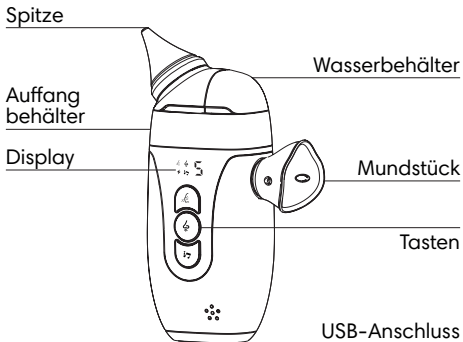Model:	VG-BCI-025
Operating temperature:	5°C and 35°C
Battery capacity:	1500 mAh
Runtime:	+/- 2.5 hours
Charging voltage:	5V
Input current:	1A
Power type:	USB Type-C

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